



**Disposable blue Velcro ties hold hose in place near top of handle and above “T Bar”.**

**Additional hose can be attached to increase length or a longer hose can be used.**

**Example: 12 to 20 feet.**

**Hoses are supplied by hospital and are universal when attaching to black “T Bar” fitting located at end.**

**Base of handle goes into “T Bar”. (Does not screw in). Rivet head found at base of handle is pushed through “T Bar” until it appears through “T Bar” Hole. Once in it cannot be removed.**

**After each use the hose is disposed of.**

**After each use the pad is disposed of.**

**After removing pad “T Bar” is bathed in yellow tray with disinfectant.**

**Once disinfected new Velcro, hose and pad can be installed for next procedure. Disinfectant is drained and replaced with fresh.**